


Revlis Sapphire™

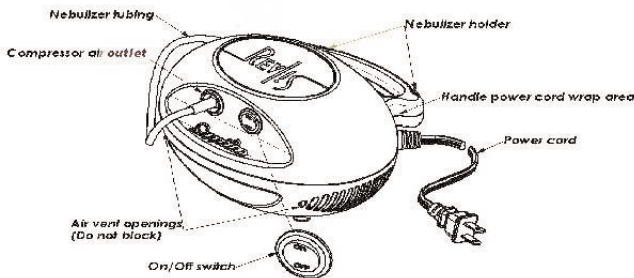

SPECIFICATIONS:

- Weight3.5 lb.
- Dimensions..... 8.5 x 8 x 5 in.
- Electrical requirements.....120V AC, 60Hz, 1.2A
- Operating temperature100 C to 400 C (500F to 1040 F)
- Storage temperature -100 C to 400 C (140F to 1040 F)
- Relative humidity.....10% to 100%

▲ DANGER


- To reduce the risk of serious or fatal injury from electrocution, fire or burns and to reduce the risk of damage and malfunction to the unit:
1. Before use, check the serial number label on the bottom of the Sapphire compressor to ensure the voltage and current indicated on the unit correspond to the voltage and current available.
 2. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been exposed to any liquids inside the case. Return the product to an authorized Revlis Service Center for examination and repair.
 3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or use extension cords.
 4. This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.
 5. Never block the air vents of this product or place it on a soft surface, such as a bed or couch, where the air vents may be blocked. Keep the air vents free of lint, hair, and similar substances. Blockage could cause excessive heat buildup. A product should never be left unattended when plugged in.
 6. Do not operate outdoors.
 7. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.
 8. Never drop or insert any object into any openings.
 9. Be sure the Revlis Sapphire™ is unplugged prior to cleaning.
 10. Do not open or disassemble the Revlis Sapphire™ compressor.
 11. Refer all servicing to an authorized Revlis Service Center.

▲ WARNING

Close supervision is necessary when this product is used by or near the physically or mentally impaired or children. Never use while sleeping or drowsy.

This device contains small parts that may present a choking hazard if swallowed. Always keep out of the reach of children three years old or under.

SAVE THESE INSTRUCTIONS

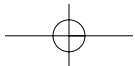

I. GETTING STARTED

1. Check the Revlis Sapphire™ packing box and make sure the following basic components are included:
 - Revlis Sapphire™ Compressor with Power Cord
 - Compressor Instruction Book
 - Nebulizer Set with Instructions For Use*

* Depending on the model purchased this package may include more than one nebulizer.

If you did not receive the components listed, please contact the dealer where you purchased your Revlis Sapphire™ Compressor Nebulizer System.

It is a good idea to always keep an extra nebulizer set on hand at all times. You may need it if parts become lost or damaged.

II. SET-UP

1. Place the Revlis Sapphire™ Compressor on a stable, flat surface. Do not place on cushioned, upholstered or carpeted surfaces. Place the unit in an area that is clean and free of dust.
2. Connect the polarized plug on the power cord to a polarized wall outlet (do not use an extension cord).
3. Open the nebulizer package and read and follow the nebulizer Instructions For Use.
4. It is important to firmly push the tubing all the way onto the air outlet of the compressor prior to starting the treatment. The tubing is designed to fit tightly.
5. Turn the power switch on to begin treatment.
6. Turn the power switch off and unplug the unit when the treatment has been completed.
7. Clean the nebulizer as per the instructions after each use.

Note: The motor has a thermal overload protector. In the event the compressor becomes overheated, it will permanently shut off the motor. Contact your home care provider.