

TRANSNEB INSTRUCTIONS

Your TransNeb is the latest in compressor Nebulizer technology. The purpose of the TransNeb is to convert your liquid medication into a fine mist that can be drawn into your lungs to relieve your respiratory condition. The TransNeb is very portable and lightweight. For even greater convenience, ask Landmark Healthcare about the optional battery pack.

Operating Instructions:

- 1) Remove lid and then remove the Nebulizer by pulling straight up.
- 2) Open the Nebulizer by pulling straight up on the top while securely holding the bottom.
- 3) Make sure the cone is correctly positioned in the cup.
- 4) Add the prescribed amount of medication to the cup. Do not exceed the maximum allowed quantity of 6cc of medication.
- 5) Close the Nebulizer by firmly pressing the top down onto the cup until a click is heard.
- 6) Replace the Nebulizer back into the compressor making sure it is firmly seated and oriented so the outlet is against the filter housing.
- 7) Rotate the Nebulizer clockwise until the desired treatment position is reached.
- 8) Connect the mouthpiece or optional pediatric mask to the Nebulizer outlet.
- 9) Turn on your TransNeb by pushing the ON/OFF button.
- 10) Begin treatment by placing the Nebulizer mouthpiece between your teeth and close your mouth. Inhale deeply and slowly through the mouth as aerosol begins to flow. Exhale slowly.
- 11) When treatment is completed, after 5 to 10 minutes, turn off the TransNeb, unplug the power adapter remove the mouthpiece or mask from the Nebulizer and rotate the Nebulizer back to "Stand-By" position. Remove the Nebulizer and clean.

Cleaning Instructions:

- **Compressor Cleaning**

Never submerge the TransNeb compressor in water, hold under running water or spray with water or other liquids. This will result in compressor damage. Wipe the TransNeb with a clean, damp cloth every 2-3 days. This will keep the device dirt and dust free.

- **Nebulizer Cleaning**

The Nebulizer is designed for single patient use. The Nebulizer should be replaced after 6 months of use. Remove the Nebulizer from the compressor or remove the tubing adapter. Remove the mouthpiece or mask. Open the Nebulizer and remove the cone from the cup. Boil the components in water for a maximum of 5 minutes. Place the components on a clean, lint free towel to air dry.

- **Filter Care**

The filter must be replaced after one month of use, or when it turns gray. The filter should be replaced if the TransNeb has been in storage for 3 months or longer.

WARNING: To prevent possible risk of infection due to contamination, clean the Nebulizer after each aerosol treatment.